

# Getting Started with Juvasonic®

enhances topical absorption | penetrates exosomes, growth factors, peptides, PRP + PDRN |  
pain-free, needle-free, no downtime | efficient use of high-grade topicals  
Use this guide to perform a complete, results-driven procedure.

## SUPPLIES CHECKLIST

Pre-Treatment	Serum Delivery	Post-Treatment
<ul style="list-style-type: none"><li>• Alcohol wipes</li><li>• Hypochlorous acid</li><li>• Gloves</li><li>• 4x4 gauze</li><li>• Headband/towel</li></ul>	<ul style="list-style-type: none"><li>• Juvasonic® (charged)</li><li>• 3 mL vial + nanotip</li><li>• 3 cc syringe</li><li>• 18G blunt tip needle</li><li>• Sterile saline</li><li>• Serum (Exosomes, PDRN, PRP, PRF, etc.)</li></ul>	<ul style="list-style-type: none"><li>• Hyaluronic acid</li><li>• Soothing mask (optional)</li><li>• SPF 30+ sunscreen</li></ul>

## STEP-BY-STEP PROCEDURE

1. Charge your Juvasonic® device.
2. Set up clean treatment area with all supplies.
3. Clean treatment area with alcohol and hypochlorous acid. Let dry.
4. Pre-condition skin/scalp with empty cartridge using vertical, horizontal, and circular motions for ~15-20 seconds per 2x2 area.
5. Withdraw serum with syringe. Dilute with saline, if needed.
6. Fill Juvasonic® 3 mL vial + nanotip.
7. Insert cartridge. Select speed. Activate red/blue LED (double tap).
8. Apply serum using same motion technique, working in sections.
9. Finish with hyaluronic acid, optional mask, and sunscreen.

### Contact Us

- (877) 676-8875
- [hello@juvasonic.co](mailto:hello@juvasonic.co)
- [www.juvasonic.co](http://www.juvasonic.co)

